MANUALE D'USO

Droppy-s

MANUALE D'USO
INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRAUCHSANLEITUNG
MANUAL DE USO

12 mesi-months-mois-monaten-meses

nome-name-nom-vorname-nombre

cognome-surname-prenom-nachname-apellido

indirizzo-address-adresse-auschrift-direccion

città-town-ville-ort-ciudad

SN _____ CS _____

data d'acquisto-purchase date-date d'achat
einkaufdatum-fecha de compra

Distillatore Droppy-s

versione-version-version-modell-versión

Timbro del rivenditore-Dealer's stamp-Cachet d'achat
Stempel der Fachhändlers-Sello del revendedor

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SAFETY REGULATIONS

- Leave the distiller with a dry, flat, heat-tolerant surface, and keep it away from any heat source.
- The distiller also needs a well ventilate space for cooling fan uses air to cool it.
- Do not share the same outlet with other electrical appliance to prevent from overloading.
- Do not use an extension cord.
- Do not pour boiling water into the collection water tank.
- Do not remove the distiller lid when it shows on.
Before removing the lid to refill, wait at least 20 minutes after the distillation is completed (use the handle attached to the lid).
- Unplug the main power cord when filling, cleaning, or not using the distiller.
- Never soak any part of the distiller in water.
Keep the outside part dry.
- Unplug the main power cord if you wish to stop the distillation process earlier.
- Keep it away from little kids.

COMPONENTS

Distill water maker

Power switch

Water collection tank

Protective cover

Activated charcoal

Residue cleaner

Power code

CHARACTERISTICS

Droppy-s machine for water distillation through vapour and activated carbons. The water obtained by distillation is indicated to be used with sterilizers in the dental field

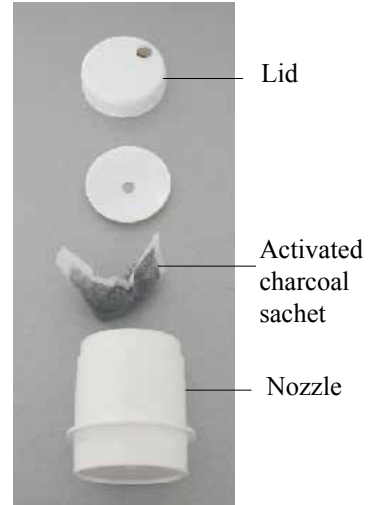
INSTRUCTION FOR USE

A. Install activated charcoal sachet

1. Soak a new activated charcoal sachet in water.
2. Turn the bottle lid with the hole, and unscrew it from the nozzle compartment.
3. Fold the activated charcoal sachet in half, and put it into the nozzle.
4. Reassemble the lid.
5. Put the lid with the hole onto the collection bottle, lining it up with the spout on the distiller.

NOTE: Each activated charcoal sachet can be used for up to a month.

Please replace it regularly. The activated charcoal dust is harmless and functions as removing odors.

B. Filling the distiller with water

1. Pour water into the distiller boiling chamber and be careful not to fill over the internal "fill line"(about 5cm from the top).
2. If you use hot water from a kettle, the total distillation time shall be reduced.

INSTRUCTION FOR USE**C. To distill water**

1. Pull the upper cover from the boiling chamber, allowing for the plug.
2. Pour water into the stainless steel boiling chamber. Make sure not to fill past the line marked inside the boiling chamber.
3. Place the upper cover on the distiller. Be sure that any spillage outside the distiller is dried off, and the electrical sockets are dry before plugging the power cord into the socket on the distiller.
4. Line up the lid with the hole on the collection bottle with the distiller nozzle. Insert the main power cord into the power socket, and turn the power switch on. You will hear the fan start to function immediately.
5. The START button on the distiller is set by the factory at "ON" position.
6. For subsequent cycles, please press firmly on the START button when distillation starts.
7. About an hour after you switch on power, distilled water will begin to drip into the storage tank.
8. The power supply shall switch off automatically when distillation is completed. Never remove the upper cover while the distiller is on. Turn off power supply if wish to stop the distillation process.

MAINTENANCE

- Never use abrasive cleaners or brushes on your distiller
- After each cycle rinse the inside of the boiling chamber with tap water and dry carefully. Do keep the outside dry.
- To remove the buildup of deposits, follow the instructions on the distiller cleaning crystals enclosed. Use it once a week or more, depends on the scale buildup in the boiling chamber.
- If it appears to have a peeling stainless steel, it needs a clean right away.
- The activated charcoal sachet should be replaced once a month or earlier.
- If the condensing coil is clogged with dust, the efficiency of distillation may be affected. To clean the condensing coil, vacuum the coil regularly through the slots in the upper cover.

GUIDE TO PROBLEMS

Why my distiller is not making any water?

- Please check the main power cord and make sure it is plugged in as well as the upper cover cord is plugged in the base of distiller

My distiller is not making enough water during each cycle, and steam is leaking from spout, why?

- Please make sure:
 - 1- Raw water is filled to the mark line inside the boiling pot
 - 2- Be sure the proper sitting of the upper gasket.
 - 3- Do place the water collection tank directly under the spout.
 - 4- Condensing fins clean and not blocked with dust.
 - 5- Fan in working
 - 6- Charcoal sachets are placed in the nozzle properly.

TECHNICAL SPECIFICATIONS

- Reduces lead, arsenic, nitrates, sodium and chromium VI
- Reduces 99.9% of Total Dissolved Solids (TDS).
- Water purification by steam distillation & carbon filtration
- Power: 240 VAC - 50Hz - 580W
- Distillate capacity approx. 3.2 liters in 4.5 hours
- Automatic shut-off after each cycle
- Stainless steel boiler with 4 liters capacity
- Stainless steel condenser
- 20cm(L) x 40 cm(W)x 36cm(H)
- Net weight 7.75lbs (3.5 Kgs)

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GUARANTEE CERTIFICATE

FARO grants to the final customer a **12-months** guarantee starting from the date of the purchase invoice. Repairs under the guarantee shall be done at the **FARO premises or by an authorized FARO Service** person with transportation costs and risks at the expense of the purchaser. The guarantee certificate found on the endleaf of this booklet is considered valid only if it is entirely completed and bears the reseller's stamp. Repairs under guarantee (during the period of effectiveness) will be done only if the article to be repaired is accompanied by the shipping note or purchase invoice. The guarantee covers failures due to poor material quality or manufacturing defects and in case of legitimate claim the guarantee will allow repair or replacement free of charge. **No compensation for damages and/or interest will be recognized.** The guarantee is not deemed valid in the final judgement of FARO in case of tampering, damage, incorrect use, faulty maintenance or normal wear.