03


SELLA II

MANUAL SEALER
SIGILLATRICE

INSTRUCTION MANUAL
MANUALE D'USO

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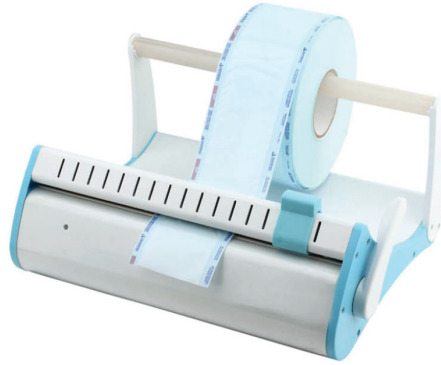1 INTENDED USE

This product is intended for sealing envelopes for steam sterilization.

Follow strictly the instructions and orientations from the manufacturer of your packages regarding their appropriate usage, cautions, conservation and storage.

2 SAFETY WARNINGS

1 - Risk of un-effective sealing: Keep the sealing platform always clean. The accumulation of waste on the sealing area will reduce the life cycle of the resistance and the insulating material. For an effective cleaning, wait for the sealing area to cool down and use a cloth slightly moistened with alcohol.

2 - Risk of electrical shock: If it is necessary to replace the fuse, use only those which have the same specifications as the genuine parts.

3 - Risk of un-effective sealing: while packing the instruments protect the ones which have cutting and piercing tips with cotton or gauze to avoid the packages rupture.

4 - Risk of un-effective sealing: Place the sealed envelopes with instruments into the autoclave. Check the instructions from the manufacturer of your autoclave regarding the arrangement of the packages and the sterilization cycle.

5 - Risk of electrical shock: always unplug the power cord when servicing or cleaning the equipment. Service must be performed by qualified personnel.

6 - Risk of electrical shock: do not spray or spill any liquid on the sealer;

7 - Risk of hot parts: Do not put fingers and other parts of the body near the resistance while the equipment is switched on.

8 - Risk of cutting: when removing the blades for replacement always pay attention to the sharp edge.

9 - Risk of instability: always install the sealer taking care that is securely supported on the shelf. Maximum dimensions are defined into the Technical data Section

10 - Risk of breakage of parts: do not pull the sealer arm with exaggerate pressure. The arm could break.

11 - Risk of un-effective pack: if the blade knob is pulled slowly the paper may crease and the cutting not adequate. To make it more precise pull the blade knob quickly. If necessary, slide the cutting blade more than once.

3 USER INSTRUCTIONS

- 1 - Mount the Holder support (see fig 1) to the chassis.
- 2 - Connect the power cord to the power connector which is on the left side of the sealer, check if the voltage of your equipment matches the voltage of the local electrical network.
- 2 - Switch the sealer on by using the power switch on its left side. When the green LED goes on (after around 3-5 minutes) the sealer will be ready to be used.
- 3 - Place the roll on the holder. The beginning of the roll shall be inserted through the back side, between the pressure bar and the sealing bar always with the paper side down (resistance's) side and the plastic side up (directly under the upper pressure bar). Push the sealer arm (4) towards Paper Holder (1) and press it down softly until it is locked.
- 4 - When the GREEN led turn to RED the sealing will be completed. An acoustic signal will be emitted to inform that the seal is ready.
- 5 - Leaving the Sealer Arm pressed cut the envelope sliding the blade (3). Raise the sealer arm and remove the envelope. If the Sealer Arm is left in "sealing" position for a longer time the RED light will start to blink and another acoustic signal will be emitted.
- 5 - For the cleaning always use a soft cloth. Never spill or spray liquids directly on the sealer.
- 6 - The smell of burnt of the equipment at the first putting into service is a normal situation. The smell disappears with the use of the equipment.

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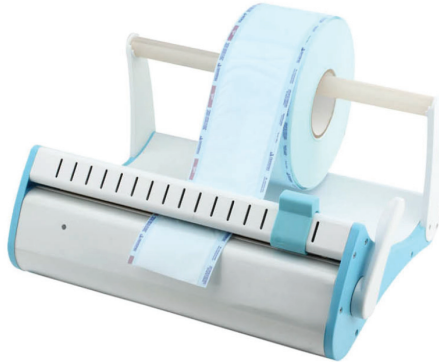
GUARANTEE CERTIFICATE

GB

Faro offers the final customer a **12 month** guarantee starting from the date of purchase.

Repairs under guarantee must be performed at FARO; expenses and transport risks are at the risk of the purchaser.

Repair under guarantee is considered valid only when:

- **all sections of the certificate have been filled in and sent in advance to FARO by Fax (039.6010540).**

The guarantee covers faults due to the bad quality of the material or manufacturing defects; in the case of valid claims, the guarantee covers free repair or replacement. **Claims for damages and/or interest are excluded.** The guarantee is not considered valid, at the sole discretion of FARO, if the fault is due to tampering, damage, incorrect use, improper maintenance and normal wear and tear.

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