04

SYR3

SIRINGA DENTALE
DENTAL SYRINGE

MANUALE D'USO
INSTRUCTION MANUAL
MODE D'EMPLOI
GEBRAUCHSANLEITUNG
MANUAL DE USO

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135°C
}}}

SAFETY RULES**GB**

The SYR3 dental syringe is designed exclusively for use in dentistry, by dental professionals or an assistant under a dentist's supervision, in order to perform specific cleaning and drying operations on the operating area. The device must be installed on a specific supply system (dental unit) and be connected to a dedicated FARO hose. The device must be sterilised in an autoclave before it is used on another patient. The device is not supplied sterile and must therefore be sterilised prior to use. Perform only the operations contained in the following manual. In all other cases request technical assistance.

HOSE AND SYRINGE INSTALLATION

Installation must be performed by specialised personnel only.

Check that the package contains the following components:

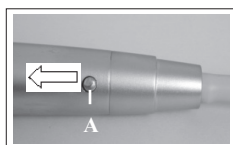
- Dental syringe.
- Supply hose.
- O-rings.
- Probe.
- Instruction manual.

Connect the BLUE tube to the AIR intake, the GREEN tube to the H₂O intake and fasten the safety cord.


Connect the hose to the syringe by tightening the ring nut. Check the water and air pressure values and make sure the fluids are at ambient temperature (Tmax < 41°C).

INSTRUCTIONS FOR USE

- For water, press the green button.
- For air, press the blue button.
- For spray, press both buttons simultaneously.

Disassembling sheath and nozzle

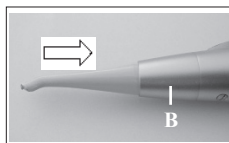- To remove the sheath, press button "A" and pull off in the direction of the arrow.

- To remove the nozzle, unscrew ring "B" and pull off in the direction of the arrow.

- Before reassembling the syringe, grease the two seals on the inner body and the metal tube of the nozzle with petroleum jelly.

Cleaning the device

Clean the sheath and the nozzle under running water or use alcohol-based solutions.

Sterilisation

- Pull the sheath, complete with the nozzle, from the body of the syringe.
- Clean sheath and nozzle.
- Sterilise the sheath complete with the nozzle in an autoclave. The sheath has a valve that prevents contamination of the body of the syringe.

Cleaning nozzle holes

- If the tiny holes in the nozzle that the water and air come out of are plugged, free the by using the specially provided probe.

TECHNICAL SPECIFICATIONS**Three-function syringe SYR3**

- Air pressure :4 bar
- Water pressure :2 bar
- Air flow :10 l/min
- Water flow :80-100 ml/min
- Blue tube for passage of :AIR
- Green tube for passage of :WATER
- Weight :100 g
- WATER-AIR temperature system temperature
- Storage with packaging intact for a maximum of 15 weeks with:
 - Ambient temperature from -20°C to +70°C
 - Relative humidity from 10% to 90%
 - Atmospheric pressure from 500 to 1060 mbar.

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GUARANTEE CERTIFICATE

FARO grants to the final customer a **12-months** guarantee starting from the date of the purchase invoice. Repairs under the guarantee shall be done at the **FARO premises or by an authorized FARO Service** person with transportation costs and risks at the expense of the purchaser. The guarantee certificate found on the endleaf of this booklet is considered valid only if it is entirely completed and bears the reseller's stamp. Repairs under guarantee (during the period of effectiveness) will be done only if the article to be repaired is accompanied by the shipping note or purchase invoice. The guarantee covers failures due to poor material quality or manufacturing defects and in case of legitimate claim the guarantee will allow repair or replacement free of charge. **No compensation for damages and/or interest will be recognized.** The guarantee is not deemed valid in the final judgement of FARO in case of tampering, damage, incorrect use, faulty maintenance or normal wear.

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FARO S.p.A.
FARO SpA si riserva il diritto di modificare, senza preavviso, le caratteristiche indicate nel presente manuale.
FARO SpA reserves the right to change the specifications of this equipment without notice.
FARO SpA se reserve le droit de modifier, sans préavis, les caractéristiques dans ce manuel.
FARO SpA behält sich recht vor, jederzeit stillschweigend technische oder bauliche Änderung vorzunehmen.
FARO SpA se reserva el derecho de modificar sin aviso previo la características incluidas en el presente manual de uso.