


1. SIDÈREA BKL WIFI CONNECTION PROCEDURE

1. Start Wifi search
2. Connecting to the Siderea BKL network
3. Enter password: Faro1948

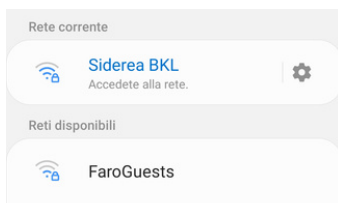

The web page appears where you can set various parameters such as:

- on/off times for the whole week of the ambient light.
- Start time for the whole week with corresponding cycle duration for BKL function
- Electronic Board usage counters
- Counters for different ambient light scenarios.
- Counter of disinfection hours
- Indirect light illuminant can be switched on and off.


How to connect Sidèrea BKL to the studio's wifi network STATIC IP SETTING

Configuring a static IP address allows the user to connect the lamp to a predefined network.
N.B. It is recommended to have this activity carried out by experienced IT personnel.

In mobile mode click on




A. Click on **Setup**

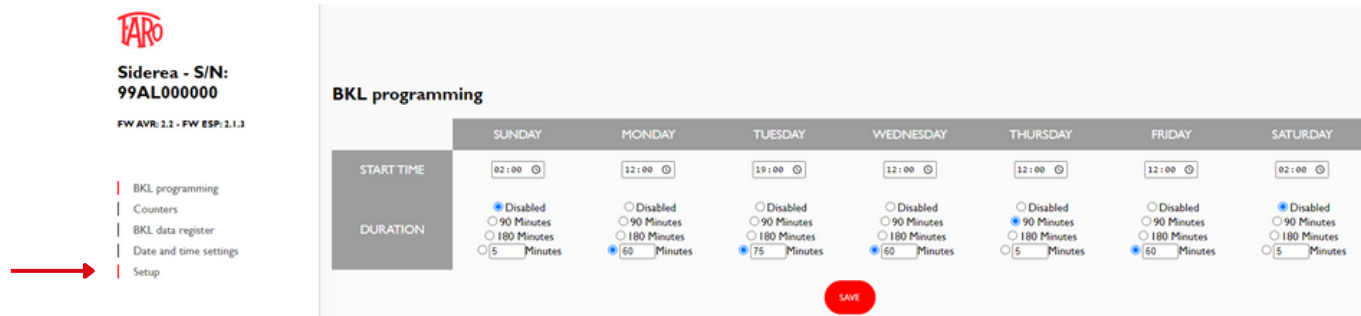


The following page is displayed:

User: admin - Password admin


B. Click on the top right menu and select "Configure new AP"

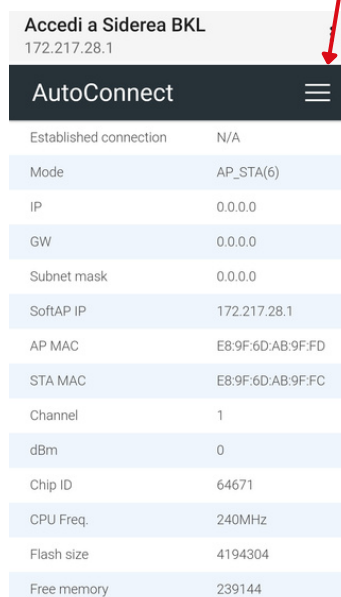

C. Disable "Enable DHCP"

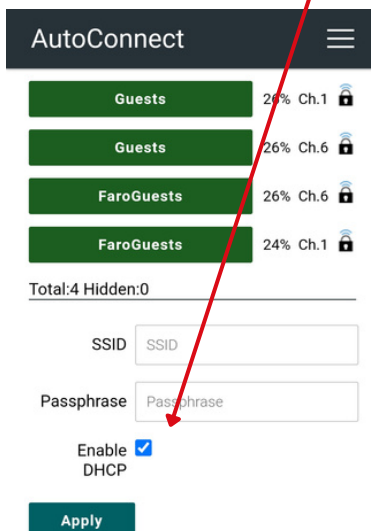

D. Enter the required data on the next screen

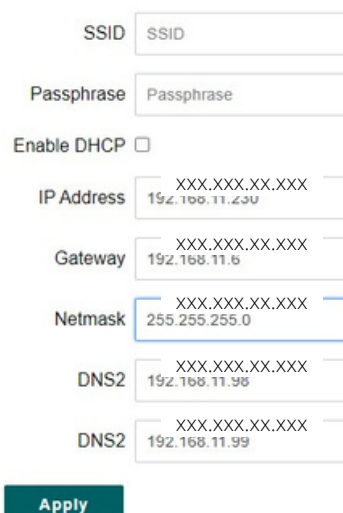



EXPERIENCE AND INNOVATION
SINCE 1948

2.